




Ozone Syringe Filling System

Regeno3oneVet.com
Email: info@regeno3onevet.com



Ozone Syringe Filling System

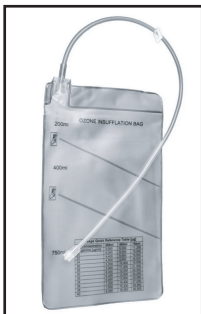



CE





Table of Contents

Page

 Ozone Syringe Filling System for the Stallion	1
 Ozone Syringe Filling System for the Woofer	2



Ozone Syringe Filling System for the Stallion

Filling Syringes With Ozone

1. Turn on medical ozone generator with switch on the back of the generator.
2. Connect syringe directly to the ozone out port on your medical ozone generator.
3. Set the desired concentration. Make sure dial is on destruct.
4. Push the ozone button on a green light will illuminate.
5. Turn the dial to 03 ozone start filling attached syringe.
6. When syringe is full turn dial back to destruct.
7. Push the ozone button off.
8. Turn off regulator, and machine.





Ozone Syringe Filling System for the Woofer

Filling Syringes With Ozone

1. Remove the Woofer from the carry case or packaging and set on a flat surface.
2. Connect one end of the 5ft piece of tubing to the oxygen regulator. Connect the other end of this tubing to the oxygen inlet.
3. Connect the 3-way valve to the ozone outlet.
4. Connect one end of the short piece of tubing provided to the side outlet of the 3-way valve. Connect the other end of this tubing to the ozone destruct inlet. Make sure the word "OFF" on the dial of the 3-way valve is pointing toward the open port on the 3-way valve, which faces outward.
5. Plug the power cord into the power inlet. A red light will illuminate.
6. Open the oxygen tank and make sure the oxygen regulator is set to "0".
7. Turn the power knob to the right to allow the generator to warm up for one minute. A green light will illuminate.
8. Turn the power knob back to the "OFF" position.
9. Connect the accessory (syringe, insufflation bag, etc.) you will be using to the available port facing outward on the 3-way valve.
10. Using the chart on the Woofer, turn the power knob to the correct level (I, II, or III).
11. Using the chart on the Woofer, turn the oxygen regulator to the correct setting.
12. Turn the dial on the 3-way valve so the word "OFF" is pointed toward the destruct tubing.
13. Once there is enough ozone in the syringe, insufflation bag, etc., turn the dial on the 3-way valve so the word "OFF" is pointed toward the syringe, insufflation bag, etc. This will stop the flow of oxygen/ozone going into the syringe, insufflation bag, etc.
14. Turn the power knob to the "OFF" position.
15. Turn the oxygen regulator to "0".
16. Unplug the generator.
17. Close the oxygen tank.
18. Turn off regulator, and machine.