

Regeno3oneVet.com Email: info@regeno3onevet.com



# Pet Ozone Spa Shower User's Manual



CE



Regeno3oneVet.com Email: info@regeno3onevet.com

| Contents                                | Page |
|---|------|
| Product Introduction & Principle        | 1    |
| Ozone Pet Spa Therapy                   | 1    |
| Treatment Options                       | 2    |
| Benefits of Ozone Hydro-Massage Therapy | 2    |
| How It Works:                           | 3    |
| Recommended Treatment Programs          | 3    |
| Product Parameter                       | 4    |
| Caution Before Operating                | 4    |
| Content & Structure                     | 5    |
| Installation Instructions               | 6    |
| Usage                                   | 7    |
| Control & Indicators                    | 7    |
| Maintenance Service                     | 8    |
| Product Registration                    | 9    |



Regeno3oneVet.com Email: info@regeno3onevet.com

#### Product Introduction & Principle

This Pet Ozone Spa Shower is a smart appliance designed for water taps. When you turn on the tap, it immediately produces ozonated water, and when you turn it off, the unit stops automatically. It is an eco-friendly solution for kitchen cleaning, hand washing, pet sterilization, and more. The shower generates ozone, a naturally occurring substance used for centuries and is approved by the EPA and FDA for use in food safety. The end cap, or mixer, infuses the ozone into the water, effectively eliminating bacteria, viruses, fungi, toxins, and other harmful organisms—without the need for harsh or costly chemicals.

#### Pet Ozone Spa Shower

\* Veterinarians \* Pet Groomers \* Pet Guardians

Pet Ozone Spa Shower offers your pet advanced spa treatments by combining the natural elements of heat, water, and air. This therapy uses hydro-massage and ozone to reduce stress, ease fatigue, relieve joint and muscle tension, and enhance skin and coat health. Ozone, a powerful oxidizer, effectively neutralizes bacteria, mold, yeast, organic matter, and viruses.



Regeno3oneVet.com Email: info@regeno3onevet.com

## **Treatment Options**

- Revitalizing Treatment: A general wellness boost to restore vitality, ideal for nervous or traumatized dogs.
- Relaxing Treatment: Designed to alleviate stress and fatigue, perfect for active, show, or working dogs.
- Circulation Treatment: Stimulates blood flow, especially beneficial for older or less active pets.
- Reactivating Treatment: Promotes skin health and reduces shedding, ideal for pets with skin issues or excessive shedding.
- Anti-Stress Treatment: A calming therapy for dogs undergoing environmental changes or hormonal imbalances.

# Benefits of Ozone Hydro-Massage Therapy

- Skin and Coat Care: Ozone's antibacterial properties help treat various skin issues like seborrhea, alopecia, yeast infections, and dermatitis. It also soothes itching, hot spots, eczema, and reduces heavy shedding.
- Full-Body Massage: Therapeutic bubbles stimulate tissues, relieve tension, promote circulation, and enhance the production of red and white blood cells, helping your dog relax and rejuvenate.
- Deep Cleansing: The bubbles gently cleanse your pet's skin, removing dirt and grease while nourishing skin tissues for improved breathability.



Regeno3oneVet.com Email: info@regeno3onevet.com

## How It Works

During the Pet Ozone Spa Shower, your pet is immersed in soothing ozonated water.

- Hydrotherapy: Ozone is infused into the water, creating bubbles that massage the skin, loosen dirt, and relax muscles.
- Topical Therapy: The soothing ozone water helps to increase and boost circulation, aiding in the healing of tired muscles, tendons, and joints while ozone detoxifies by killing and neutralizing bacteria and viruses.

## Recommended Treatment Programs:

| Skin Conditions:   | 1 bath per week      |
|--------------------|----------------------|
| Relaxation:        | 1 bath every 2 weeks |
| Heavy Shedding:    | 2 baths per month    |
| Stress Relief:     | 1 bath per month     |
| Elderly Pets:      | 1 bath every 2 weeks |
| Weight Management: | 1 bath every 2 weeks |



Regeno3oneVet.com Email: info@regeno3onevet.com

### **Product Parameter**

| Technical specification table (WPI0.5-D) |  |  |  |
|--|--|--|--|
| Model                                    | Ozone Shower Water                             |  |  |
| Electrical                               | InputAC100-240V, 50/60Hz<br>OutputDC 12V, <12W |  |  |
| Ozone Concentration                      | 0.5~1.0 mg/ L                                  |  |  |
| For Tap Water Flow                       | 4-8 L/min                                      |  |  |
| For Tap Water Pressure                   | 0.14~0.7 Mpa (20 - 100psi)                     |  |  |
| Operate Temperature                      | 0 °c ~ 6 0 °c                                  |  |  |
| Product Dimension                        | 223*162.5*54.5mm                               |  |  |
| Net Weight                               | 0.790 Kgs                                      |  |  |

# Caution Before Operating

Read the User's Manual carefully before operating.

- ★ Mount the unit to the wall at a position with a higher water level than the water tap.
- \* Do not let any water go into the unit.
- Use the new silicone hose attached when connecting the main body to the water tap.
- Minimum water inlet pressure is 0.14Mpa and water flow rate 4L/min.
- Maximum water inlet pressure is 0.7Mpa (~100psi)

- Do not operate the unit when the cover is removed.
- Disconnect ALL power before maintenances.
- ★ Make sure the air inlet on the back side and other ventilation holes are unimpeded
- Do not insert any metal objects into the unit.
- Do not fasten or loosen the universal water tap adapter many times.
- ★ Use a soft cloth wet with water. Do not clean this appliance with harsh chemicals, abrasives or cleansers.



Regeno3oneVet.com Email: info@regeno3onevet.com

## Content & Structure





Regeno3oneVet.com Email: info@regeno3onevet.com

#### Installation Instruction



- Unpack the box and ensure all components are present.
- Connect the power adaptor cable to the main body at the back side.
- Mount the main body to the wall. Make sure the main body is slightly higher than the water faucet, within 1 meter from the water faucet.
- Remove existing aerator from water faucet spout.
- Assemble the ozone mixer in different ways according to different water taps,
  - A: if water tap with 22mm outer thread, assemble the ozone mixer
  - B: if water tap with 24mm inner thread, assemble the ozone mixer with a 24 to 22mm thread adaptor.

If the water tap doesn't have threads, use the universal adapter to connect the ozone mixer. First, dry the faucet using a tissue or cloth to remove any moisture. Fit the universal adapter onto the faucet spout. If the spout is too large, you can remove the inner silicone layer from the universal adapter. Silde the screw ring up and tighten it with a flathead screwdriver. Attach the ozone mixer (faucet aerator) to the universal adapter by turning it clockwise, but avoid over-tightening.

Next, connect the silicone tube to the aerator tube, and attach the other end of the tube to the ozone outlet on the main unit. You can trim the silicone tube to the desired length for a neat setup.

Finally, plug the power adapter into a 100-240V AC outlet. You will hear a sound when the power is connected, and both indicator lights will turn on.



Regeno3oneVet.com Email: info@regeno3onevet.com

## Usage

- Simply turn on the water tap, and the unit will activate instantly. You'll hear a sound and see the LCD screen begin counting the working time in minutes and seconds, indicating that ozonated water is being produced.
- 2) If you prefer regular water, press the on/off button. You will hear a sound, and the LCD screen will display 4 dots to show the ozone function is off.
- $\cdot 3$ ) To switch back to ozonated water, just press the button again.

# Control & Indicators

| Diagram               | Actual Picture | Description                      | Indication |
|-----------------------|----------------|----------------------------------|------------|
| ON                    | 8.8.8.8.       | Two points                       | Standby    |
| 8.8 <sup>1</sup> 8.8. | ©              | ON                               |            |
| MM:SS<br>8.8-8.8      | 88:88<br>©     | MM:SS<br>(minute &<br>second) ON | Working    |
| 8.8:88.               | 8.8.8.8.       | Four points                      | Stop by    |
| ON                    | ©              | ON                               | Manual     |



Regeno3oneVet.com Email: info@regeno3onevet.com

# MAINTENANCE SERVICE

- This appliance is warranted to be free from all defects in material and workmanship in normal household use for a period of (1) year from date of purchase.
- The warranty is granted only to the original purchaser and members of his or her immediate household. The warranty is subject to the following provisions. Any damages or malfunctions caused by negligence, abuse, or not operate in accordance with the User's Manual are not covered by this warranty. Likewise, any defects or damages caused by unauthorized service or the use of other than genuine our specified parts are not covered.
- Our company will, at its option, repair or replace a defective appliance or part(s) covered by this warranty. As a matter of warranty policy, our company will not refund the customer's purchase price. Please record the following Warranty Card when you purchase the unit from our company or the distributor.
- To obtain warranty service you must return the defective appliance along with proof of purchase to Regeno3one Vet. All transportation charges on parts, or units, submitted under this Warranty shall be borne by purchaser. Unless this Warranty is expressly renewed or extended by our company, any repaired or replaced part of unit shall be warranted to the original purchaser only for the length of the unexpired portion of the original warranty.



Regeno3oneVet.com Email: info@regeno3onevet.com

PRODUCT REGISTRATION Thank you for choosing Regeno3one Vet's Product. To register your product, please fill in the details below and email back to us.

| First Name             | Last Name     |
|------------------------|---------------|
|                        |               |
|                        |               |
|                        | ۱ <u>۲</u>    |
| Company                | 1             |
|                        |               |
|                        |               |
| Email Address          | Phone Number  |
|                        |               |
|                        |               |
| Charact Address        |               |
| Street Address         |               |
|                        |               |
|                        |               |
| Apartment, suite, etc. | ·             |
|                        |               |
|                        |               |
| City                   | State         |
|                        |               |
|                        |               |
|                        |               |
| Zip Code               | Country       |
|                        |               |
|                        |               |
| Model Name             | Serial Number |
|                        |               |
| Pet Ozone Spa Shower   |               |
|                        |               |
| Purchase From          | Purchase Date |
|                        |               |
|                        | 11 1          |